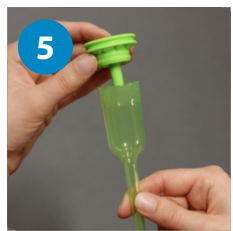
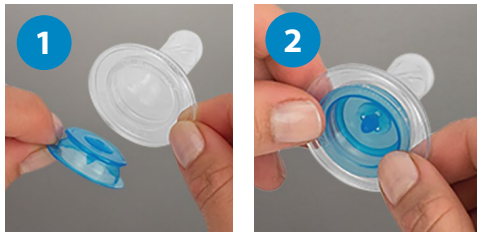
Bottle Assembly

Zero-Resistance™ Specialty Feeding System Bottle Assembly:

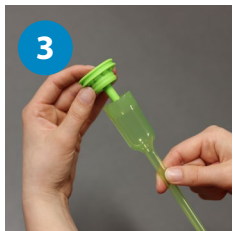
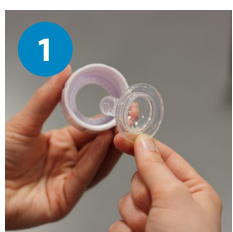
1. Insert the Infant-Paced Valve (blue one-way valve) into the nipple/teat.
 2. Make sure the valve is fully secured, flush with the nipple/teat.
 3. Insert the nipple/teat into the collar.
 4. Make sure the nipple/teat is fully seated in the collar.
 5. Assemble the Internal Vent System by placing the insert into the reservoir to snugly fit together.
 6. Place the assembled Internal Vent System into the bottle.
 7. Ensure the insert makes full contact with the top of the bottle.
- Note: If warming the liquid in the bottle, do not complete this step until after the warming is complete to prevent leakage.**
8. Affix the collar to the bottle and turn until closed (do not over-tighten).
- Note: With the Infant-Paced Feeding Valve (blue one-way valve) in place, the nipple/teat will fill with fluid as infant elevates tongue to underside of the nipple/teat during feeding. No manual priming of the nipple/teat is necessary.**
- Note: Use travel disk when in transit.**

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Dr. Brown's® Zero-Resistance™ Bottle Assembly:

1. Insert the nipple/teat into the collar.
 2. Make sure the nipple/teat is fully seated in the collar.
 3. Assemble the Internal Vent System by placing the insert into the reservoir to snugly fit together.
 4. Place the assembled Internal Vent System into the bottle.
 5. Ensure the insert makes full contact with the top of the bottle.
- Note: If warming the liquid in the bottle, do not complete this step until after the warming is complete to prevent leakage.**
6. Affix collar to bottle and turn until closed (do not over-tighten).
- Note: No air bubbles will be evident during sucking due to Dr. Brown's® Zero-Resistance™ Bottle System.**
- Note: Use travel disk when in transit.**

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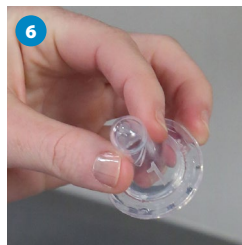
Bottle Assembly

Cleaning and Sanitizing

1. Always use clean hands and wash cleaning surfaces with solution appropriate for areas with food contact.
2. *Prior to first use*, disassemble the bottle system and clean all parts with soap and water, rinse well, and air dry. Consider sanitizing with **Dr. Brown's® Microwave Steam Sanitizing Bags** (or comparable process). *After each use*, the bottle system should be disassembled, washed with soap and water, rinsed well, and air dried.
3. Use **Dr. Brown's® Non-Metal Bottle Brush** or **AccuBrush™** soft bristles to eliminate debris from the interior of the Dr. Brown's bottle/vessel and collar.
4. Use a **Dr. Brown's Wire Cleaning Brush** to thoroughly clean the internal vent system (i.e., insert and reservoir tube) and Infant-Paced Feeding Valve (blue one-way valve) only. **Note: Hospital use is recommended to replace the wire cleaning brush every 24 hours after use. Do not use wire cleaning brush to clean nipples/teats.**
5. Use the chevron tip at the bottom of either nonmetal bottle brush to eliminate debris inside the tip of the nipple/teat and assure the nipple/teat hole is clean and open.
6. Prior to each use and after cleaning, pull on the nipple/teat in all directions to make sure it does not tear. **Note: Replace the nipple/teat prior to feeding if the integrity of the nipple/teat is compromised.**
7. Consider sanitizing/sterilizing the Dr. Brown's bottle system (disassembled), nonmetal brushes, and one piece silicone pacifiers at least every 24 hours after cleaning thoroughly with soap and water. **Note: Do not place wire cleaning brush into the Microwave Steam Sanitizing Bag.**
8. Place 2 oz/60mL of water into bag and seal bag tightly. Each Dr. Brown's® Microwave Steam Sanitizing Bag can be used for a total of 30 times at the power levels and time frames indicated on the bag.
9. Use care when removing bag from microwave as contents are hot. Cool for 10 minutes. Use drain spout to pour out excess water before removing sanitized parts. **Note: Follow the directions on the side of the bag and observe the "WARNING-SAFETY NOTE" to avoid injury when using.**

***Steam Sterilization (Autoclaving) – All Dr. Brown's® bottle systems and nipples have been tested up to 30 cycles of autoclaving. (Letter of validation is available if required – Letter to support Steam Sterilization – Autoclave)**

NOTE: The Dr. Brown's® Non-Metal Bottle Brush and AccuBrush™ cannot be Steam Sterilized as the material for the products will not sustain the high temperature. Dr. Brown's® Company has not tested the Specialty Feeding System Infant-Paced Feeding Valve or Pacifiers with Steam Sterilization.

For questions, concerns, or additional information regarding the assembly, cleaning, and sanitization of Dr. Brown's® Medical products, please contact a clinical liaison at medinfo@drbrownsmmedical.com.

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