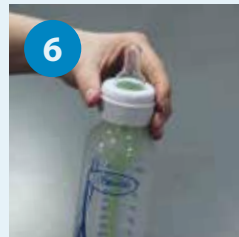
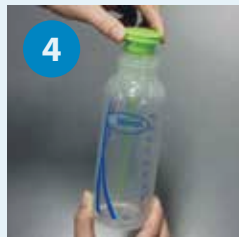
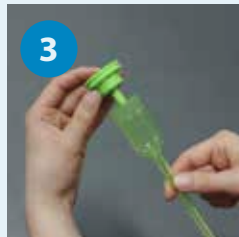
## Zero-Resistance™ Bottle Assembly

Assemblage du biberon de Dr. Brown's® Zero-Resistance™  
Montaje del biberón Dr. Brown's® Zero-Resistance™

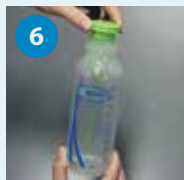
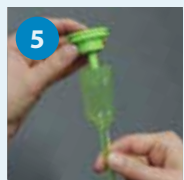
1. Insert the nipple/teat into the collar.
2. Make sure the nipple/teat is fully seated in the collar.
3. Assemble the Internal Vent System by placing the insert into the reservoir to snugly fit together.
4. Place the assembled Internal Vent System into the bottle.  
**Note: If warming the liquid in the bottle, do not complete steps 4-6 until after the warming is complete to prevent leakage.**
5. Ensure the insert makes full contact with the top of the bottle.
6. Affix collar to bottle and turn until closed (do not over-tighten).  
**Note: Due to internal vent system, no air bubbles will be evident during sucking.**

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## Specialty Feeding System Bottle Assembly

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Specialty Feeding System includes the Infant-Paced Feeding Valve (blue one-way valve) creating a "compression" nipple/teat.

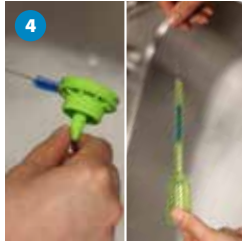
1. Insert the Infant-Paced Feeding Valve (blue one-way valve) into the nipple/teat.  
**Note: For Wide-Neck, it may be preferable to insert nipple into collar before inserting Infant-Paced Feeding Valve.**
2. Make sure the valve is fully secured, flush with the nipple/teat base.
3. Insert the nipple/teat into the collar.
4. Make sure the nipple/teat is fully seated in the collar.
5. Assemble the Internal Vent System by placing the insert into the reservoir to snugly fit together.
6. Place the assembled Internal Vent System into the bottle.  
**Note: If warming the liquid in the bottle, do not complete steps 6-8 until after the warming is complete to prevent leakage.**
7. Ensure the insert makes full contact with the top of the bottle.
8. Affix collar to bottle and turn until closed (do not over-tighten).  
**Note: With the Infant-Paced Feeding Valve (blue one-way valve) in place, the nipple/teat will fill with fluid as infant elevates tongue to underside of the nipple/teat during feeding. No manual priming of the nipple/teat is necessary.**

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## Cleaning and Sanitizing

1. Use clean hands and clean surfaces when handling. **Note: In hospital/clinical setting, consider wearing protective gloves.**
2. **Before first use and after each use**, disassemble and wash all parts with soap and water, rinse well, and air dry.
3. Use **Dr. Brown's® Non-Metal Bottle Brush** or **AccuBrush™** soft bristles to eliminate debris from the interior of the Dr. Brown's bottle/vessel and collar (or comparable process).
4. Use a **Dr. Brown's Wire Cleaning Brush** to thoroughly clean the internal vent system (i.e., insert and reservoir tube) and Infant-Paced Feeding Valve (blue one-way valve) only. **Note: Hospital use is recommended to replace the wire cleaning brush every 24 hours after use.**

**WARNING: Do not use wire cleaning brush to clean nipples/teats as this will cause the nipple to tear.**

5. Use the chevron tip at the bottom of either nonmetal bottle brush to eliminate debris inside the tip of the nipple/teat and assure the nipple/teat hole is clean and open.
6. Prior to each use and after cleaning, pull on the nipple/teat in all directions to make sure it does not tear. **Replace the nipple/teat if the integrity is compromised.**

**WARNING: Do not microwave wire cleaning brush. Follow instructions printed on the bag and observe the "WARNING-SAFETY NOTE" to avoid injury.**

**Steam Sterilization (Autoclaving) – All Dr. Brown's bottle systems, Infant-Paced Feeding Valve, and nipples have been tested up to 30 cycles of autoclaving. (Letter of validation is available if required – Letter to support Steam Sterilization – Autoclave)**

**WARNING: The Dr. Brown's Non-Metal Bottle Brush and AccuBrush cannot be steam sterilized as the material for the products will not sustain the high temperature.**

For questions, concerns, or additional information regarding the assembly, cleaning, and sanitization of Dr. Brown's Medical products, please email: [medinfo@drbrownsmc.com](mailto:medinfo@drbrownsmc.com).

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